

Definition and Technical Assessment of the Non-Damaging Sterilization Boundary and Validation Protocol for Flexible Endoscopes Using Hdrozone

Hasan Tahsin ÖZBEK • Bülent DEVECİ • Yüksel ERGÜN

White Paper - **16.10.2025**

EXECUTIVE SUMMARY

Flexible endoscopes are multi-layered hybrid medical devices consisting of elastomers, fluoropolymers, metals, and optical components. This composite structure simultaneously necessitates both microbiological efficacy and the preservation of material integrity throughout the sterilization process.

This study evaluates the applicability of the Hdrozone sterilization approach for flexible endoscopes in terms of material compatibility, lumen penetration, functional integrity, and residual safety. It also establishes the validation framework for defining the non-damaging sterilization window.






KEYS

Sterilization, Hdrozone, Lumen, Penetration, Compatibility, Residuals, Oxidation, Endoscope

PROBLEM STATEMENT

ENDOSCOPE CROSS SECTION

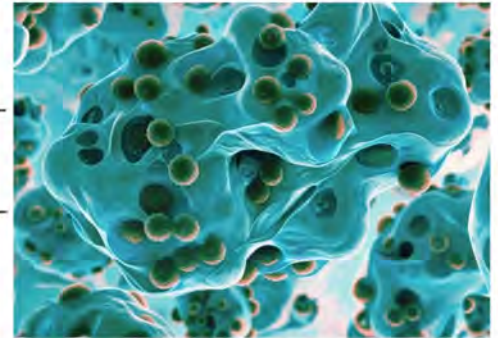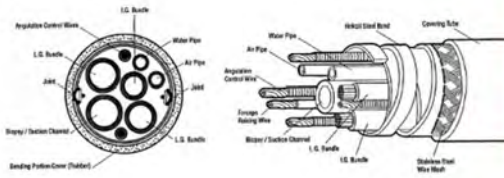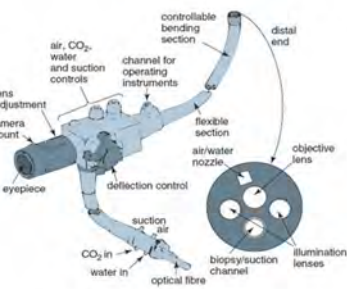

The fundamental challenge in flexible endoscope sterilization is that the device is not a homogeneous structure.

A flexible endoscope is a multi-layered system comprising:

- Outer sheath polymers
- Inner spiral metal structure
- Flexible distal tip
- Internal lumen channels
- Optical system
- Adhesive and epoxy joints

Due to this structure, the sterilization challenge involves:

- Not merely surface sterilization
- Required access to internal lumen surfaces
- Presence of biofilm and organic residues
- Varying chemical resistance of different materials

Therefore, the sterilization process simultaneously demands:

- Achieving microbiological efficacy
- Preventing material damage
- Avoiding functional degradation
- Maintaining optical quality
- Ensuring residual safety

DEFINITION AND TECHNICAL FRAMEWORK

The Hydrozone sterilization approach is a method targeting low-temperature sterilization through the controlled use of gas-phase oxidant systems.



The technical objectives of this approach are:

- Achieving target microbiological efficacy
- Preserving material, functional, and optical integrity
- Remaining within safe residual limits

The objective within this framework is to define the non-damaging sterilization process window for flexible endoscopes.

This approach is consistent with reprocessing validation requirements for reusable medical devices.

ENDOSCOPE STRUCTURE AND MATERIAL ANALYSIS

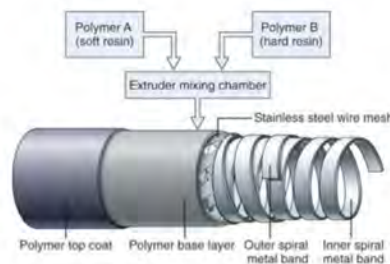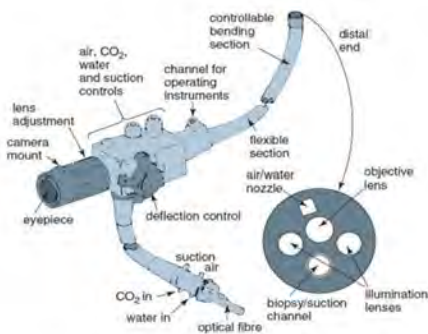

A flexible endoscope consists of the following key components:

- TPU / silicone / Pebax outer sheath
- PTFE / FEP / ETFE lumen channels
- Stainless steel spiral structure
- Nitinol support elements
- Optical fiber or lens system
- Epoxy and medical-grade adhesives

Within this structure:

- PTFE and metal components are chemically more resistant
- TPU, silicone, and epoxy regions are chemically more sensitive

Hypothesis

When the Hydrozone cycle is operated in a regime characterized by:

- High oxidant predominance
- Controlled radical activity
- Low temperature
- Controlled humidity

It can establish a non-damaging sterilization window within acceptable material change limits for flexible endoscopes.

This statement constitutes a hypothesis requiring experimental validation.

VALIDATION APPROACH AND SAMPLE PLAN

The validation approach consists of three levels:

- Coupon tests
- Sub-system tests
- Full device tests

Coupon Tests

- TPU
- Silicone
- Pebax
- PTFE / FEP / ETFE
- Epoxy
- Lens adhesive
- Seal elastomers

Sub-System Tests

- Insertion tube
- Bending section
 - Distal tip
- Working channel

Full Device Tests

- Baseline reference
 - Nominal cycle
 - Worst-case cycle
 - Aged cycle

a minimum of n=10 controls and multiple dose groups shall be applied.

Cycle Plan

- 1x
- 10x
- 25x
- 50x
- 100x

Test Groups

- Control
- Low oxidative load
- Nominal process
- High oxidative load
 - Humidity effect
 - Time effect
- Cycle repetition effect
- Plasma-free control

Measurement Methods

Mechanical	Chemical	Functional	Residual	Microbiological
<ul style="list-style-type: none"> • Hardness • Tensile strength • Bending • Torsion • Leak test 	<ul style="list-style-type: none"> • FTIR spectroscopy • Contact angle measurement • Mass change • Surface analysis 	<ul style="list-style-type: none"> • Channel flow rate • Optical quality • Image performance 	<ul style="list-style-type: none"> • H₂O₂ / O₃ residuals • Off-gassing 	<ul style="list-style-type: none"> • BI / CI • Internal lumen testing • Distal tip testing

Acceptance Criteria

- Leak test shall pass
- No visible damage
- Optical quality shall be maintained
- Channel flow rate shall remain unchanged
- Adhesive integrity shall be preserved
- Residuals shall be below defined limits
- Microbiological efficacy shall be achieved

Non-Damage Decision System

- Pass
- Conditional pass
- Fail



Damage Index (DI)

$$DI = 0.25 \times \text{Mechanical} + 0.20 \times \text{Optical} + 0.20 \times \text{Chemical} + 0.20 \times \text{Functional} + 0.15 \times \text{Residual}$$

Worst-Case Definition

- Longest lumen
- Narrowest channel
- Most sensitive polymer
- Highest humidity
- Maximum cycle count

H₂O₂ & HDROZONE COMPARISON

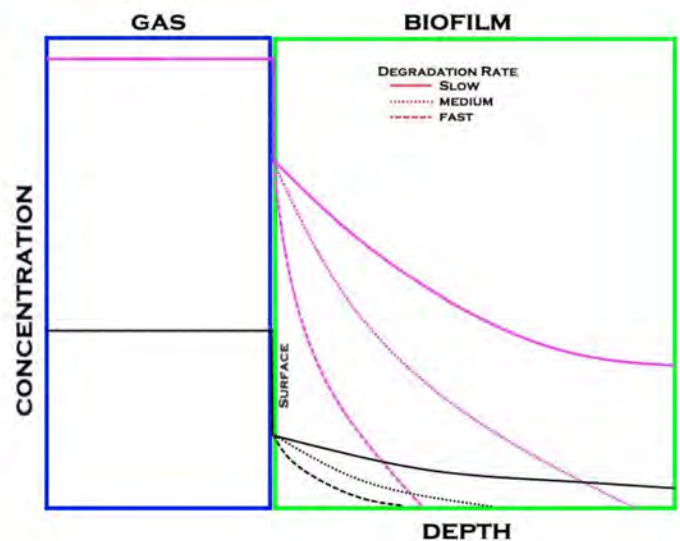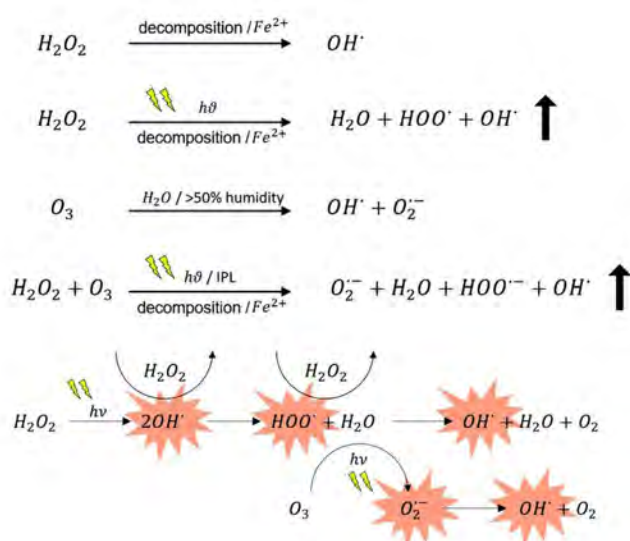

► **H₂O₂ Sterilization:**

- Strong oxidative effect at the surface
- Condensation-based mechanism
- Lumen penetration may be limited

► **Hdrozone Sterilization:**

- Gas-phase distribution
- Reactive species generation approach
- Efficacy potential on internal lumen surfaces

► **Material Perspective:**

- H₂O₂: Higher stress on epoxy and elastomers
- Hdrozone: Controlled chemical impact

TECHNICAL OUTCOME ASSESSMENT

- PTFE, FEP, ETFE: Compatible
- Metal structure: Compatible
- Optical glass: Compatible
- TPU: Critical
- Silicone: Critical
- Epoxy: Critical
- Distal tip: Critical

Sterilization success is determined by the performance on internal lumen surfaces.

CONCLUSION

Flexible endoscope sterilization represents one of the most challenging sterilization problems due to material diversity and lumen geometry.

► **The Hdrozone approach is technically a viable sterilization method owing to:**

- Low-temperature operation
- Gas-phase delivery
- Lumen penetration potential

► **Compared to H₂O₂ systems, it offers:**

- Similar surface efficacy
- Higher potential within lumens

► **Final suitability is determined through:**

- Material compatibility testing
- Cyclic aging
- Lumen penetration validation
- Microbiological efficacy testing

OFFICIAL CONCLUSION STATEMENT

This study was prepared with the objective of assessing whether the Hdrozone sterilization cycle can achieve microbiological efficacy in flexible endoscopes while remaining within acceptable limits with respect to material integrity, functional performance, optical quality, and residual safety, and to establish the technical framework for defining the non-damaging sterilization process window.

REFERENCES

Bu Daniela Trogolo, J. Samuel Arey, Peter R. Tentscher. Gas-Phase Ozone Reactions with a Structurally Diverse Set of Molecules: Barrier Heights and Reaction Energies Evaluated by Coupled Cluster and Density Functional Theory Calculations. *The Journal of Physical Chemistry A* 2019, 123 (2) , 517-536. DOI: 10.1021/acs.jpca.8b10323.

1. Alexandra Fischbacher, Katja Löppenberg, Clemens von Sonntag, and Torsten C. Schmidt . A New Reaction Pathway for Bromite to Bromate in the Ozonation of Bromide. *Environmental Science & Technology* 2015, 49 (19) , 11714-11720. DOI: 10.1021/acs.est.5b02634.

2. Yi Yang, Jin Jiang, inglin Lu, Jun Ma, and Yongze Liu . Production of Sulfate Radical and Hydroxyl Radical by Reaction of Ozone with Peroxymonosulfate: A Novel Advanced Oxidation Process. *Environmental Science & Technology* 2015, 49 (12) , 7330-7339. DOI: 10.1021/es506362e.

3. Janez Cerkovnik and Božo Plesničar . Recent Advances in the Chemistry of Hydrogen Trioxide (HOOH). *Chemical Reviews* 2013, 113 (10) , 7930-7951. DOI: 10.1021/cr300512s.

4. Alexandra Fischbacher, Justus von Sonntag, Clemens von Sonntag, and Torsten C. Schmidt . The •OH Radical Yield in the H₂O₂ + O₃ (Peroxone) Reaction. *Environmental Science & Technology* 2013, 47 (17) , 9959-9964. DOI: 10.1021/es402305r.

5. Carlos Barrera-Díaz, Lina. A. Bernal-Martínez, Reyna Natividad, and Juan M. Peralta-Hernández . Synergy of Electrochemical/O₃ Process with Aluminum Electrodes in Industrial Wastewater Treatment. *Industrial & Engineering Chemistry Research* 2012, 51 (27) , 9335-9342. DOI: 10.1021/ie3004144.

6. Sergej Naumov and Clemens von Sonntag . Standard Gibbs Free Energies of Reactions of Ozone with Free Radicals in Aqueous Solution: Quantum-Chemical Calculations. *Environmental Science & Technology* 2011, 45 (21) , 9195-9204. DOI: 10.1021/es2018658.

7. J. Pablo Pocosales, Myint M. Sein, Wolfgang Knolle, Clemens von Sonntag, and Torsten C. Schmidt. Degradation of Ozone-Refractory Organic Phosphates in Wastewater by Ozone and Ozone/Hydrogen Peroxide (Peroxone): The Role of Ozone Consumption by Dissolved Organic Matter. *Environmental Science & Technology* 2010, 44 (21) , 8248-8253. DOI: 10.1021/es1018288.

8. Helena Jablonowski, Joao Santos Sousa, Klaus-Dieter Weltmann, Kristian Wende, Stephan Reuter. Quantification of the ozone and singlet delta oxygen produced in gas and liquid phases by a non-thermal atmospheric plasma with relevance for medical treatment. *Scientific Reports* 2018, 8 (1) DOI: 10.1038/s41598-018-30483-w.

9. Yalei Ding, Jiejie Wang, Shanshan. u, Kun-Yi Andrew Lin, Shaoping Tong. Oxygen vacancy of CeO₂ improved efficiency of H₂O₂/O₃ for the degradation of acetic acid in acidic solutions. *Separation and Purification Technology* 2018, 207, 92-98. DOI: 10.1016/j.seppur.2018.06.027.

10. Donggwan Lee, Jae-Cheol Lee, Joo-Youn Nam, Hyun-Woo Kim. Degradation of sulfonamide antibiotics and their intermediates toxicity in an aeration-assisted non-thermal plasma while treating strong wastewater. *Chemosphere* 2018, 209, 901-907. DOI: 10.1016/j.chemosphere.2018.06.125.
11. David B. Miklos, Christian Remy, Martin Jekel, Karl G. Linden, Jörg E. Drewes, Uwe Hübner. Evaluation of advanced oxidation processes for water and wastewater treatment – A critical review. *Water Research* 2018, 139, 118-131. DOI: 10.1016/j.watres.2018.03.042.
12. Pierre-François Biard, Thom Thi Dang, Jenny Bocanegra, Annabelle Couvert. Intensification of the O₃/H₂O₂ advanced oxidation process using a continuous tubular reactor filled with static mixers: Proof of concept. *Chemical Engineering Journal* 2018, 344, 574-582. DOI: 10.1016/j.cej.2018.03.112.
13. Emmanuel Mousset, Nihal Oturan, Mehmet A. Oturan. An unprecedented route of OH radical reactivity evidenced by an electrocatalytical process: Ipso-substitution with perhalogenocarbon compounds. *Applied Catalysis B: Environmental* 2018, 226, 135-146. DOI: 10.1016/j.apcatb.2017.12.028.
14. J.F. Pérez, S. Sabatino, A. Galia, M.A. Rodrigo, J. Llanos, C. Sáez, O. Scialdone. Effect of air pressure on the electro-Fenton process at carbon felt electrodes. *Electrochimica Acta* 2018, 273, 447-453. DOI: 10.1016/j.electacta.2018.04.031.
15. Guilherme Garcia Bessegato, João Carlos de Souza, Juliano Carvalho Cardoso, Maria Valnice Boldrin Zanoni. Assessment of several advanced oxidation processes applied in the treatment of environmental concern constituents from a real hair dye wastewater. *Journal of Environmental Chemical Engineering* 2018, 6 (2) , 2794-2802. DOI: 10.1016/j.jece.2018.04.041.
16. A. Privat-Maldonado, Y. Gorbanev, D. O'Connell, R. Vann, V. Chechik, M. W. van der Woude. Nontarget Biomolecules Alter Macromolecular Changes Induced by Bactericidal Low-Temperature Plasma. *IEEE Transactions on Radiation and Plasma Medical Sciences* 2018, 2 (2) , 121-128. DOI: 10.1109/TRPMS.2017.2761405.
17. Mika Sillanpää, Mohamed Chaker Ncibi, Anu Matilainen. Advanced oxidation processes for the removal of natural organic matter from drinking water sources: A comprehensive review. *Journal of Environmental Management* 2018, 208, 56-76. DOI: 10.1016/j.jenvman.2017.12.009.
18. B.W. Darvell. *More Chemistry*. 2018,, 771-789. DOI: 10.1016/B978-0-08-101035-8.50030-4.
19. Erika Reisz, Clemens von Sonntag, Agnes Tekle-Röttering, Sergej Naumov, Winfried Schmidt, Torsten C. Schmidt. Reaction of 2-propanol with ozone in aqueous media. *Water Research* 2018, 128, 171-182. DOI: 10.1016/j.watres.2017.10.035.
20. João F. Gomes, Inês Leal, Katarzyna Bednarczyk, Marta Gmurek, Marek Stelmachowski, Magdalena Diak, M. Emília Quinta-Ferreira, Raquel Costa, Rosa M. Quinta-Ferreira, Rui C. Martins. Photocatalytic ozonation using doped TiO₂ catalysts for the removal of parabens in water. *Science of The Total Environment* 2017, 609, 329-340. DOI: 10.1016/j.scitotenv.2017.07.180.

21. Pierre-François Biard, Thom Thi Dang, Annabelle Couvert. Determination by reactive absorption of the rate constant of the ozone reaction with the hydroperoxide anion. *Chemical Engineering Research and Design* 2017, 127, 62-71. DOI: 10.1016/j.cherd.2017.09.004.
22. Ozge Turkay, Sibel Barışçı, Mika Sillanpää. E-peroxone process for the treatment of laundry wastewater: A case study. *Journal of Environmental Chemical Engineering* 2017, 5 (5) , 4282-4290. DOI: 10.1016/j.jece.2017.08.012.
23. Minhwan Kwon, Homin Kye, Youmi Jung, Yejoon Yoon, Joon-Wun Kang. Performance characterization and kinetic modeling of ozonation using a new method: R OH,O₃ concept. *Water Research* 2017, 122, 172-182. DOI: 10.1016/j.watres.2017.05.062.
24. Ligy Philip, Bhallamudi Murty, Channarong Puchongkawarin, Miao Guo, Nilay Shah, David Stuckey, Benoit Chachuat, Yannic Vaupel, Sarojini Tiwari, Chitta Behera, Babji Srinivasan, Chedly Tizaoui, Olajumoke Odejimi, Ayman Abdelaziz. 3 Wastewater Treatment. 2017,, 115-180. DOI: 10.1201/9781315153209-4.
25. Songjie Wu, Qian Zhang, Ruonan Ma, Shuang Yu, Kaile Wang, Jue Zhang, Jing Fang. Reactive radical-driven bacterial inactivation by hydrogen-peroxide-enhanced plasma-activated-water. *The European Physical Journal Special Topics* 2017, 226 (13) , 2887-2899. DOI: 10.1140/epjst/e2016-60330-y.
26. Julia Patzsch, Jonathan Z. Bloh. Improved photocatalytic ozone abatement over transition metal-grafted titanium dioxide. *Catalysis Today* 2017, DOI: 10.1016/j.cattod.2017.07.010.
27. Xinyang Li, Shaobin Sun,, u Zhang, Guicheng Liu, Clark Renjun Zheng, Jianzhong Zheng, Dayi Zhang, Hong Yao. Combined electro-catazone/electro-peroxone process for rapid and effective Rhodamine B degradation. *Separation and Purification Technology* 2017, 178, 189-192. DOI: 10.1016/j.seppur.2016.12.052.
28. Jingxin Yang, Ji Li, Wenyi Dong, Jun Ma, Jiayin Li. Influence of nitrite on the degradation of atrazine by ozonation. *Journal of Chemical Technology & Biotechnology* 2017, 92 (2) , 442-450. DOI: 10.1002/jctb.5031.
29. George Wafula Wanjala, Arnold Onyango, Calvin Onyango, Moses Makayoto. . *African Journal of Biochemistry Research* 2017,, 79. DOI: 10.5897/AJBR2017.0967.
30. Glen Andrew de Vera, Wolfgang Gernjak, Howard Weinberg, Maria José Farré, Jurg Keller, Urs von Gunten. Kinetics and mechanisms of nitrate and ammonium formation during ozonation of dissolved organic nitrogen. *Water Research* 2017, 108, 451-461. DOI: 10.1016/j.watres.2016.10.021.
31. M. E. Zappi, R. Hernandez, D. Gang, R. Bajpai, C. H. Kuo, D. O. Hill. Treatment of groundwater contaminated with high levels of explosives using advanced oxidation processes. *International Journal of Environmental Science and Technology* 2016, 13 (12) , 2767-2778. DOI: 10.1007/s13762-016-1109-x.
32. Juhong Zhan, Yujue Wang, Huijiao Wang, Wenhua Shen,, uejun Pan, Jinlin Wang, Gang Yu. Electro-peroxone regeneration of phenol-saturated activated carbon fiber: The effects of irreversible adsorption and operational parameters. *Carbon* 2016, 109, 321-330. DOI: 10.1016/j.carbon.2016.08.034.

33. Jingxin Yang, Ji Li, Wenyi Dong, Jun Ma, Jie Cao, Tingting Li, Jiayin Li, Jia Gu, Pingxin Liu. Study on enhanced degradation of atrazine by ozonation in the presence of hydroxylamine. *Journal of Hazardous Materials* 2016, 316, 110-121. DOI: 10.1016/j.jhazmat.2016.04.078.
34. Xin Cheng, Hongguang Guo, Hongwei Liu, Yang Liu, Ying Yang, Yongli Zhang. Performance and Mechanism on Degradation of Estriol Using O₃/PS Process. *Ozone: Science & Engineering* 2016, 38 (5), 358-366. DOI: 10.1080/01919512.2016.1170589.
35. László Wojnárovits, Erzsébet Takács. Radiation Induced Degradation of Organic Pollutants in Waters and Wastewaters. *Topics in Current Chemistry* 2016, 374 (4) DOI: 10.1007/s41061-016-0050-2.
36. Jaedon Shin, Zahra Ramadhany Hidayat, Yunho Lee. Influence of Seasonal Variation of Water Temperature and Dissolved Organic Matter on Ozone and OH Radical Reaction Kinetics During Ozonation of a Lake Water. *Ozone: Science & Engineering* 2016, 38, 100-114. DOI: 10.1080/01919512.2015.1079120.
37. Yury Gorbanev, Deborah O'Connell, Victor Chechik. Non-Thermal Plasma in Contact with Water: The Origin of Species. *Chemistry - A European Journal* 2016, 22 (10), 3496-3505. DOI: 10.1002/chem.201503771.
38. Arnold N. Onyango. Alternatives to the 'water oxidation pathway' of biological ozone formation. *Journal of Chemical Biology* 2016, 9, 1-8. DOI: 10.1007/s12154-015-0140-6.
39. Fei Qi, Wei Chu, Bingbing. u. Comparison of phenacetin degradation in aqueous solutions by catalytic ozonation with CuFe₂O₄ and its precursor: Surface properties, intermediates and reaction mechanisms. *Chemical Engineering Journal* 2016, 284, 28-36. DOI: 10.1016/j.cej.2015.07.095.
40. Erika Reisz, Sergej Naumov, Winfried Schmidt, Clemens von Sonntag. Reaction of Ozone with Ag(I)—Mechanistic Considerations. *Ozone: Science & Engineering* 2015, 37, 393-404. DOI: 10.1080/01919512.2015.1041583.
41. Huijiao Wang, Shi Yuan, Juhong Zhan, Yujue Wang, Gang Yu, Shubo Deng, Jun Huang, Bin Wang. Mechanisms of enhanced total organic carbon elimination from oxalic acid solutions by electro-peroxone process. *Water Research* 2015, 80, 20-29. DOI: 10.1016/j.watres.2015.05.024.
42. Jesús Ferre-Aracil, Salvador C. Cardona, Javier Navarro-Laboulais. Kinetic Study of Ozone Decay in Homogeneous Phosphate-Buffered Medium. *Ozone: Science & Engineering* 2015, 37, 330-342. DOI: 10.1080/01919512.2014.998756.
43. Fei Qi, Bingbing. u, Wei Chu. Heterogeneous catalytic ozonation of phenacetin in water using magnetic spinel ferrite as catalyst: Comparison of surface property and efficiency. *Journal of Molecular Catalysis A: Chemical* 2015, 396, 164-173. DOI: 10.1016/j.molcata.2014.10.001.
44. Erika Reisz, Alexandra Fischbacher, Sergej Naumov, Clemens von Sonntag, Torsten C. Schmidt. Hydride Transfer: A Dominating Reaction of Ozone with Tertiary Butanol and Formate Ion in Aqueous Solution. *Ozone: Science & Engineering* 2014, 36, 532-539. DOI: 10.1080/01919512.2014.891436.
45. Sergey L. Khursan. Peroxide intermediates of oxidation processes: Organic trioxides. 2014,, 1-72. DOI: 10.1002/9780470682531.pat0874.

46. S. Reed Plimpton, Mark Gołkowski, Deborah G. Mitchell, Chad Austin, Sandra S. Eaton, Gareth R. Eaton, Czesław Gołkowski, Martin Voskuil. Remote delivery of hydroxyl radicals via secondary chemistry of a nonthermal plasma effluent. *Biotechnology and Bioengineering* 2013, 110 (10.1002/bit.v110.7) , 1936-1944. DOI: 10.1002/bit.24853.
47. Zdenko Machala, Barbora Tarabova, Karol Hensel, Eva Spetlikova, Libusa Sikurova, Petr Lukes. Formation of ROS and RNS in Water Electro-Sprayed through Transient Spark Discharge in Air and their Bactericidal Effects. *Plasma Processes and Polymers* 2013, 10 (10.1002/ppap.v10.7) , 649-659. DOI: 10.1002/ppap.201200113.
48. Josep M. Anglada, Miquel Torrent-Sucarrat, Manuel F. Ruiz-Lopez, Marilia Martins-Costa. Is the HO₄⁻ Anion a Key Species in the Aqueous-Phase Decomposition of Ozone?. *Chemistry - A European Journal* 2012, 18 (10.1002/chem.v18.42) , 13435-13445. DOI: 10.1002/chem.201201991.
49. Marek Golkowski, Czesław Golkowski, Jori Leszczynski, S. Reed Plimpton, Piotr Maslowski, Aleksandra Foltynowicz, Jun Ye, Bruce McCollister. Hydrogen-Peroxide-Enhanced Nonthermal Plasma Effluent for Biomedical Applications. *IEEE Transactions on Plasma Science* 2012, 40, 1984-1991. DOI: 10.1109/TPS.2012.2200910.
50. D. Gardoni, A. Vailati, R. Canziani. Decay of Ozone in Water: A Review. *Ozone: Science & Engineering* 2012, 34, 233-242. DOI: 10.1080/01919512.2012.686354.
51. Sergej Naumov, Clemens von Sonntag. The reaction of •OH with O₂, the decay of O₃^{•-} and the pKa of HO₃[•] - interrelated questions in aqueous free-radical chemistry. *Journal of Physical Organic Chemistry* 2011, 24 (10.1002/poc.v24.7) , 600-602. DOI: 10.1002/poc.1812.
52. Rossano Amadelli, Luca Samiolo, Achille De Battisti, Alexander B. Velichenko. Electro-oxidation of Some Phenolic Compounds by Electrogenerated O₃ and by Direct Electrolysis at PbO₂ Anodes. *Journal of The Electrochemical Society* 2011, 158, P87. DOI: 10.1149/1.3589913.
53. Pentax, White Papers
54. ASP, White Papers
55. Olympus, White Papers